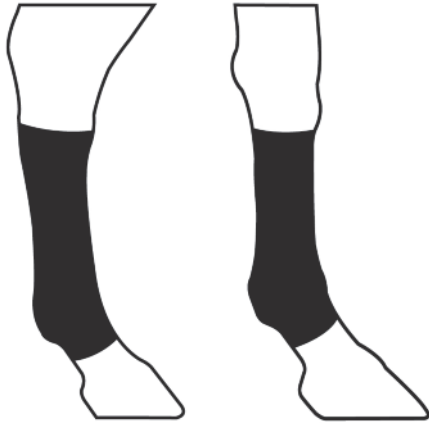
EquiCrown®

EquiCrown®

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Dear Customer,

You have selected a product that you can rely on. The name Juzo®, and the high level of technical know-how of our employees, is your

guarantee because excellent product quality has always been a tradition at Juzo®.

EN

EquiCrown® compression bandages for horses are made of a very robust and durable knitted fabric. However, avoid any contact with pointed or sharp-edged items to prevent damage to the bandages.

Donning of EquiCrown® active:

1. Place the bandage around the horse leg starting at the back of the leg. The closure should point to the outer side. The fetlock needs to be placed into the provided bulge.
2. First close the Velcro® fastening to pre-set the bandage.
3. Reopen the top Velcro® piece to fasten zipper, continue to zip the bandage downward while unfastening and then securing Velcro® over the closed zipper.
4. The end of the zipper should be folded back and secured under the Velcro® fastener.
5. If necessary adjust the bandage for fit. The fastening needs to point to the outer side of the horse leg.

Donning of EquiCrown® med/ EquiCrown® fit Silver:

1. Place the bandage around the leg of the horse starting at the back of the leg. The closure should point to the front. The bandage needs to be closed right on top of the cannon bone.
2. First fasten several hooks in the outermost 2 or 3 rows of hooks at the top end of the bandage to secure the bandage to the leg.
3. Next, starting from the bottom, fasten the hooks to the appropriate corresponding rows of hooks.
4. You may find it necessary to tighten the bandage by one row of hooks during the course of the treatment.

Please note:

- Use EquiCrown® compression bandages only for the intended indications and after consultation with your vet or animal therapist.
- As therapy progresses or the condition changes, it may be necessary to remeasure for a new EquiCrown® compression bandage.
- EquiCrown® compression bandages are mostly resistant to oils, ointments, clay, skin moisture and other environmental influences.
- Wash the EquiCrown® compression bandages regularly to maintain consistent compression.
- Store the bandages in a dry place at room temperature.
- If the bandages get damaged, please consult your dealer. Do not repair the bandages yourself as this could impair the quality and effectiveness.

Care and washing instructions

Before washing we recommend that all hooks and zippers are closed and turned inside out. This also allows for the easiest removal of dirt and horsehair. The bandages should be washed separately the first time (the dye may run). In case of daily use, EquiCrown® compression bandages should be washed weekly on a delicate cycle (40 °C). Please do not use any softener! If washing by hand, rinse thoroughly and do not wring. We recommend using Juzo®'s special mild special detergent. You can reduce the drying time by placing the washed bandage on a thick towel, rolling the towel up tightly, and firmly pressing out excess moisture. Line dry the bandages. Do not leave the bandages in the towel and do not dry on a heater or in the sun. If tumble-dry, please use the delicate cycle. Do not dry clean the EquiCrown® bandages. Please refer to the washing symbols on the label sewn into the bandages.

Material composition

Details are provided on the label sewn into the EquiCrown® compression bandage.

Indications

EquiCrown® active:

- swollen legs
- soft windgalls
- after tendon injury
- after fetlock injury
- postoperative to prevent from swellings
- postoperative and post-traumatic oedema
- post phlegmon
- scar treatment
- wound treatment in conjunction with wound dressing as substitute for medical wrap bandage
- as a preventative method during long standing periods (e.g. stall rest, transport)

EquiCrown® med (to be used only after consulting the veterinarian):
same indications as for EquiCrown® active additionally:

- (chronic) elephantiasis or extraordinary shapes of the legs
- pronounced/hardened windgalls
- after abate of phlegmon (leg still with heavy oedema)
- chronic phlegmon
- chronic lymphatic disorders
- lymphatic oedema, congestive oedema, inactivity oedema
- to supplement therapy after manual lymph drainage and compression wrap bandages
- chronic tendon disease

EquiCrown® fit Silver/med Silver:

- skin irritations (e.g. mud fever)
- prevention of phlegmon after injuries
- post injury treatment (typically in conjunction with wound dressing)

In case of a chronic phlegmon the circumference of the leg should first be reduced with lymphatic drainage. EquiCrown® med compression bandages can assist in maintaining the smaller circumference of the leg. EquiCrown® med is also available with silver fibers knit into the fabric creating an anti-bacterial effect.

Contraindications

Do not use in case of fever or during acute phlegmon. There is a possibility of bacteria entering into the circulatory system. The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

None known. However, if negative alterations (e.g. skin irritations) should occur during use, please go to contact your veterinarian or therapist immediately. If an incompatibility should be known for your horse against one or several contents of this product, please consult with your veterinarian before use. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

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